drive™

www.drivemedical.com

Rev.5.02.25.14

ready set go
shower chair

silla para ducha portátil

chaise de douche
ready set go

item # RTL12105KDR

EU Authorized Representative
Drive International, LLC
Globe Lane - Stockport
DUKINFIELD SK16 4RE, GB

www.drivemedical.com

drive™

installation instructions

WEIGHT CAPACITY 300 LBS (136 KG)

1. Turn shower chair upside down on flat surface.
2. Insert legs into shower chair. Be sure snap buttons pop out through legs completely (Figure 1).
3. Turn over shower chair with legs attached so all four legs are on the floor.
4. Slide elbow tubing into back receptacle under the seat (Figure 2). Be sure snap buttons pop out completely. Repeat for second elbow.
5. Attach back support into elbows (Figure 3). Be sure snap buttons pop out completely.

adjusting leg extensions

1. Select one (1) of the eight (5) adjustment holes located on the lower leg frame.
2. Depress the snap button and slide the adjustable leg up or down to the desired height.
3. Ensure that the snap buttons fully protrude through the same respective adjustment hole of each leg extension.
4. Repeat STEPS 2-3 for the other legs.
5. Make sure all four (4) legs are adjusted to the same height so that the shower chair sits level.

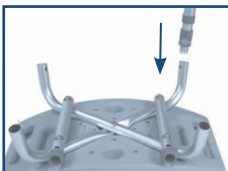
figure 1

figure 2

figure 3

lifetime limited warranty

Your Drive branded product is warranted to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

© 2014 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China

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© 2014 Medical Depot, Inc. Todos los derechos reservados.
Drive es marca registrada de medical depot, Inc.
© 2014 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.